XO FLOW

EXTRAORDINARY DENTISTRY

XO CARE empowers professionals to perform extraordinary dentistry by enhancing the practitioners' skills and protecting their health, while providing the best patient experience and ensuring high return on investment.

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NEW TRENDS

The dental industry is undergoing a significant transformation as practices become larger with many – often part-time – employees.

Practitioners request integration of the treatment units with other dental devices and software solutions.

Clinic owners demand a high return on investment.

Finally, health authorities require more standardization, documentation and increased infection control.

In response to these trends, we present XO FLOW. A unit that is easy to use, allows all users to access their individual settings from any unit in a network and enables the management to incorporate standard treatment protocols.

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XO FLOW

XO FLOW is a digital dental unit with workflow guidance and dental apps designed to help professionals perform extraordinary dentistry.

The unit is fitted with a built-in computer, network connection and other novel digital features not seen in dentistry before: the Dashboard and the Navigator.

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
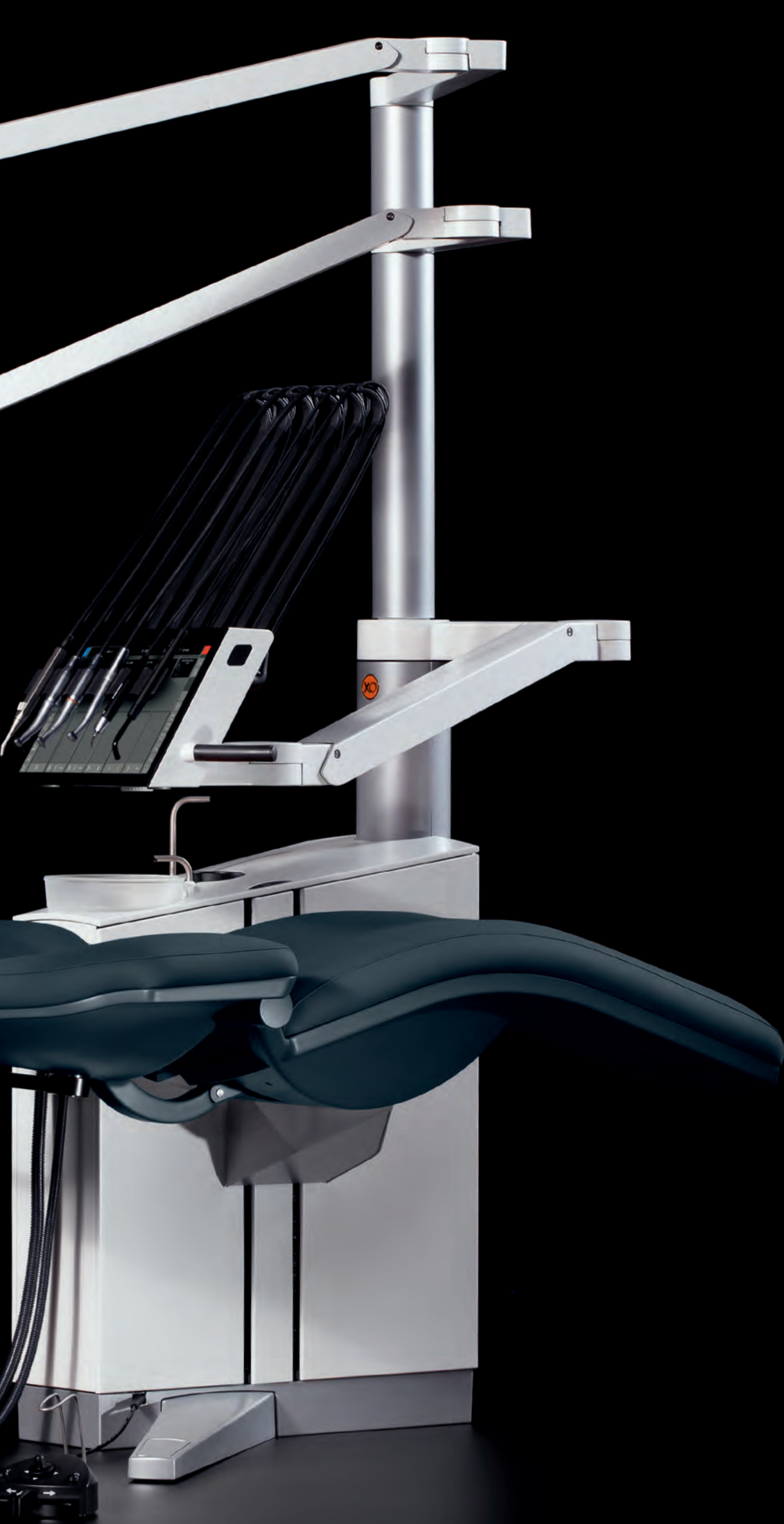
DASHBOARD

The Dashboard is a touch-activated graphical user interface integrated with the instrument bridge.

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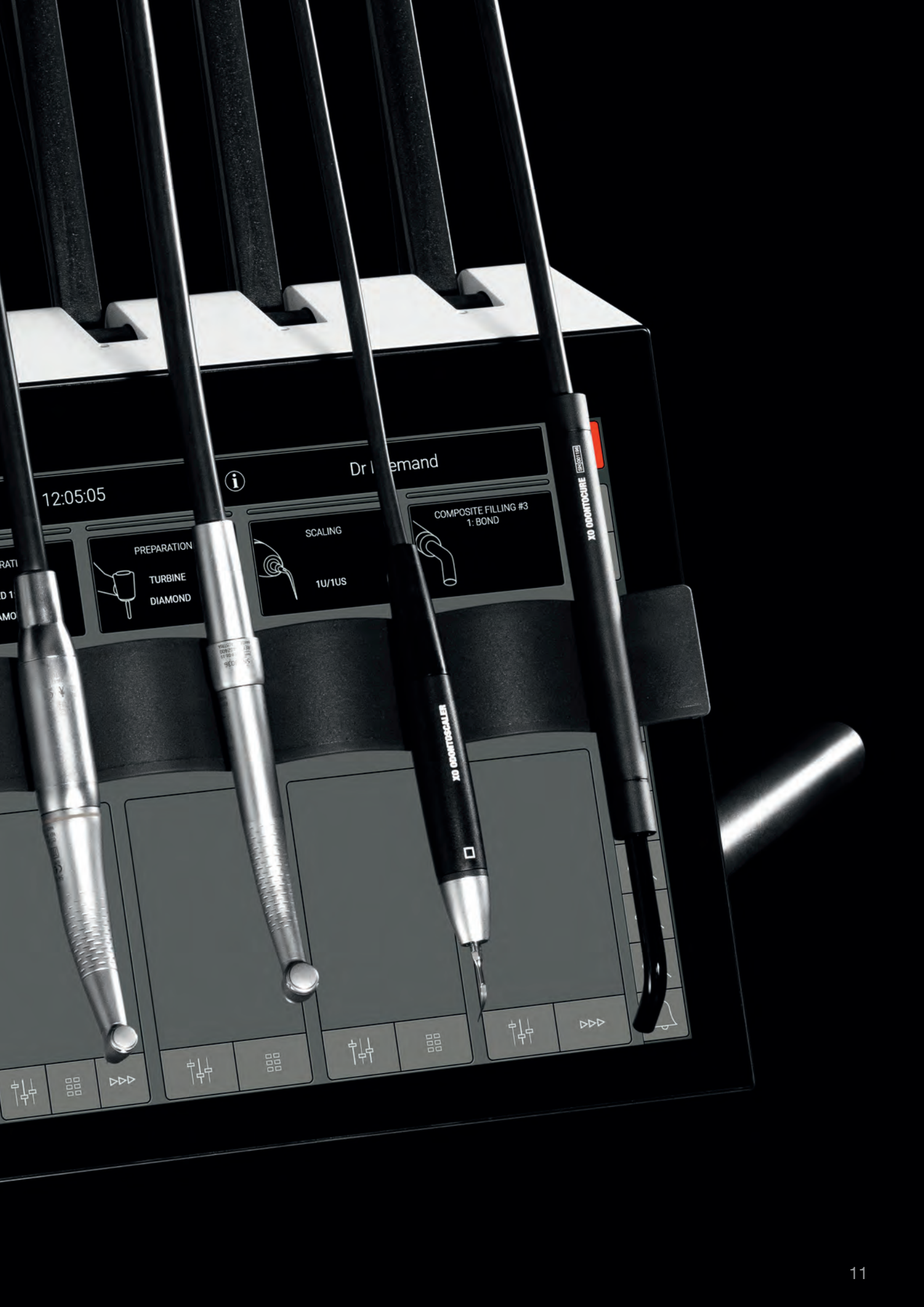XO Instrument Workflows

NAVIGATOR

The Navigator is a touch-activated screen offering a smart user interface with dental apps. The apps are used for setting up, administering, and integrating the unit with other dental devices and software solutions in a network.

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**Red Dot Winner 2021
in category
“Medical Devices and Technology”**

Statement by the jury:

“With its intuitive touchscreen technology and floating elegance, XO FLOW enhances any dental practice and bridges the gap between functionality and comfort.”

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**Red Dot Winner 2021
in category
“Innovative products”**

Statement by the jury:

“The digitalization of the dental treatment workspace implemented in XO FLOW substantially enhances and simplifies processes in dental practices.”

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ENHANCING PROFESSIONAL DENTISTRY

DASHBOARD CONTROL

Both the dentist and the chair-side assistant use the Dashboard to control and monitor all unit and instrument functions – without losing focus on the patient.

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EASY ACCESS TO IDEAL INSTRUMENT SETTINGS

The digital technology of XO FLOW makes it possible to control all instrument settings from the Dashboard.

Based on this, we have designed instrument presets that allow the operators – with just one click – to select ideal instrument settings for a specific dental procedure.

Presets also offer guidance on selection of contra angles, burs, etc., making it easy for the operators to enhance their performance.

Users can create their own presets or work with the default presets created by XO CARE in cooperation with leading dentists from around the world.

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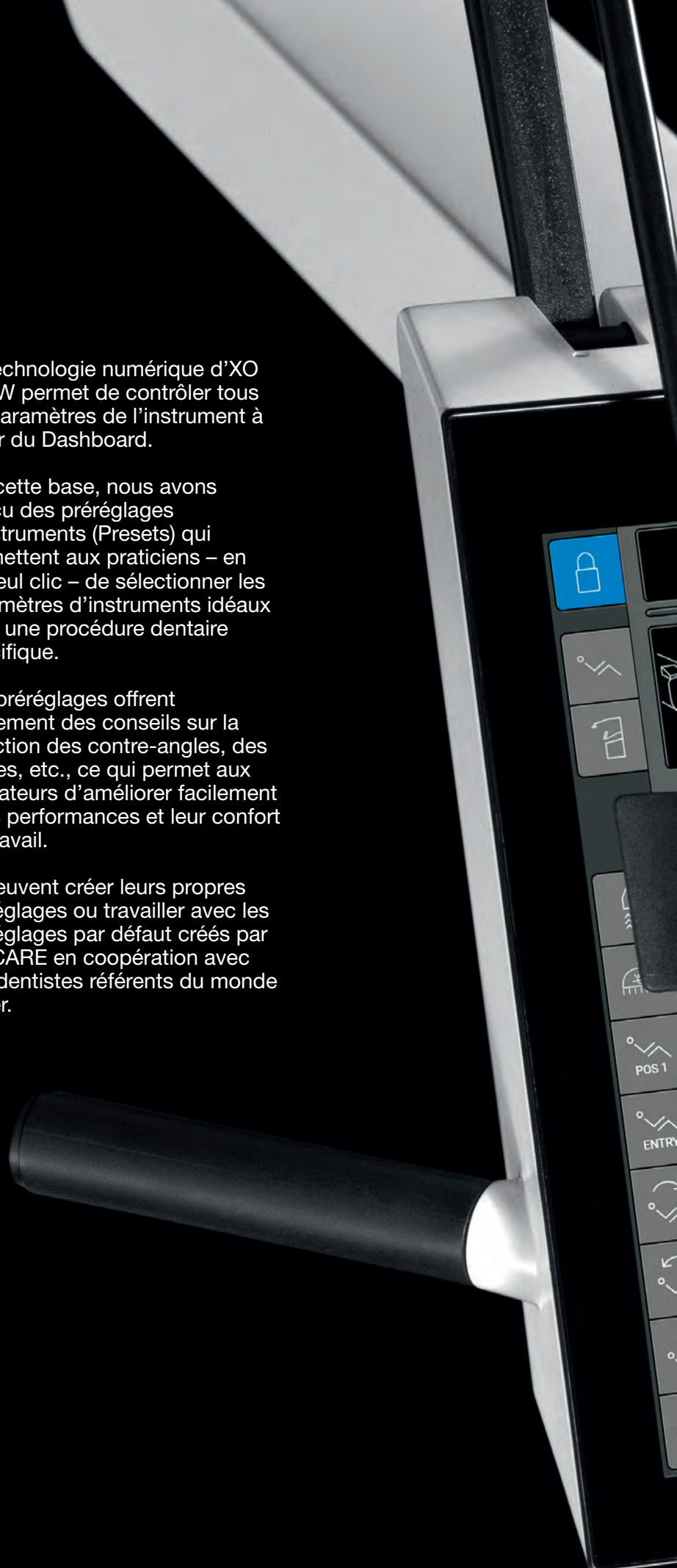
AIR INSTRUMEN

XO ODONTOSCALER

 PEDIO	 ACRYLIC
 FINISHING	 POLISHING / CLEANING
 PREPARATION	 EXCAVATION

Control buttons:   

Control buttons:   

Control buttons:  

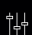
Control buttons: 

MONITOR AND ADJUST A PRESET


If necessary, the users can tap  to access sliders and buttons for adjusting speed, spray and other instrument settings.

The automatic chip blow function – which provides a constant clear eyesight without changing to/or from the syringe – is disabled/enabled with .

In the EXCAVATION preset, the “tactile function” – which makes the bur stop rotating when touching healthy dentine – may be disabled with .

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AIR INSTRUMEN

XO ODONTOSCALER

Handpiece control panel with three sliders (red, green, blue) and various icons including a hand, an eye, a refresh symbol, a stop symbol, and a play symbol.

Select workflow

Filters

Task

- Deep cavity layered
- Deep cavity bulk
- Superficial cavity
- Occlusal cavity
- Erosion/abrasion
- Bonded crown/veneer

Color

- Light
- Dark

Tooth

- Molar
- Premolar
- Anterior

Workflows

COMPOSITE FILLING +
Deep cavity layered, Molar, Light

COMPOSITE FILLING +
Molar, Deep cavity layered, Dark

COMPOSITE FILLING +
Molar, Deep cavity bulk, Light

COMPOSITE FILLING +
Molar, Deep cavity bulk, Dark

COMPOSITE FILLING +
Premolar, Deep cavity layered, Light

COMPOSITE FILLING +
Premolar, Deep cavity layered, Dark

GUIDED WORKFLOW MANAGEMENT

From the Navigator, the practitioners have access to the instrument workflow app.

This app is used to manage multi-step dental procedures like curing of composite fillings or preparation of root canals.

Based on the operator's input, the system provides a treatment protocol with optimal instrument settings for each step.

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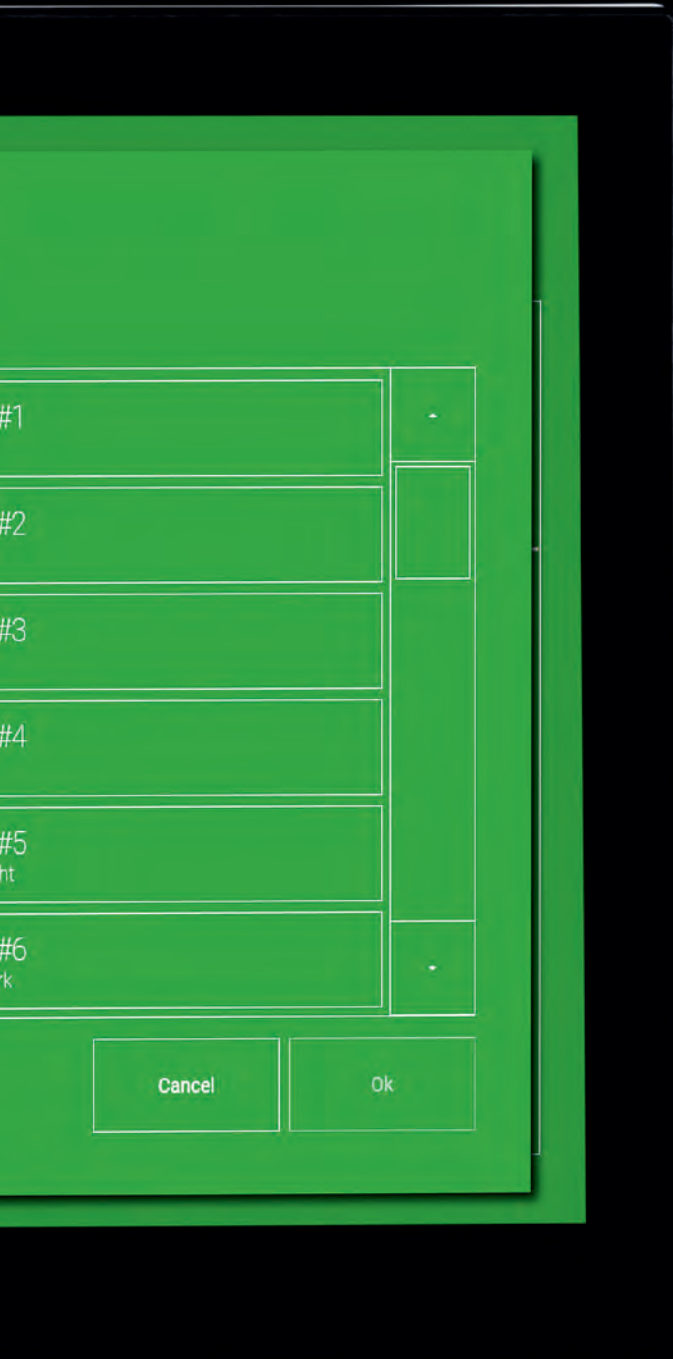
2: FLOW

SAFE WORKFLOW EXECUTION

When a workflow is activated, the Dashboard shows the workflow steps.

The first step is executed using the foot control. After that, the next step is selected, and the users continue like this until all steps have been completed.

For curing light workflows, each step sets up optimal exposure time and radiant exitance.

Workflows for root canal preparations – where each step specifies selection of file, optimal speed and torque – are set up and activated in a similar way.

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FOOT CONTROL

The foot control is used to activate instruments and to operate the patient chair. This helps the dental team to maintain focus on the patient.

It is possible to configure the foot control to function like classic XO units.

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INSTRUMENTS AND DASHBOARD WITHIN EASY REACH

The instrument bridge is centrally placed and positioned near the oral cavity close to the patient's chest. This makes it possible for the dentist and chairside assistant to see the unit instruments and the Dashboard in the peripheral field of vision. Also, they can easily reach the unit instruments and operate the Dashboard without losing focus on the patient.

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CLEVER AND CONVENIENT

Navigator home screen

The home screen shows the available dental apps.

At any time, the home button may be tapped to return to the home screen.

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Remote desktop app

The remote desktop app is used to show and interact with software running on computers accessible in the network.

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Status app

The status app gives access to information such as the date of the next preventive service and the need to replace consumables.

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Administration app

The administration app is used to set up and configure the unit. For example, to create new users.

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DENTAL APPS

Select user app

This app is used for changing the active user.

Instruments, unit and chair are automatically set up in accordance with each user's preferences.

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Camera app

The camera* app makes it possible to control an integrated intraoral video camera.

Saved images may be managed with the images* app.

*The camera app and the images app will be available 2022 at the same time as the intraoral camera module.

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Ergonomy guide app

In relation to the tooth and tooth surface to be treated, this app guides the users on healthy and optimal working positions and optimal patient positions.

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Pretreatment checklist app

This app gives guidance on how to position and use the unit, suction, light and patient chair to get maximum benefit from XO FLOW.

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FOUR NAVIGATOR POSITIONS

**PROVIDING
BEST PATIENT
EXPERIENCE**

XO FLOW offers an unequalled patient experience.

The patient has free access to the chair when the instrument bridge is in the parking position.

Silent electric motors quickly place the patient in the working position.

The unit is supplied with a patient tray where personal items such as keys and glasses can be placed during the treatment.

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When the instrument bridge is placed in the working position over the patient's chest, she neither sees nor touches the instruments as they are shielded by the bridge.

The patient chair is extremely comfortable with ideal support of the patient's neck and arms.

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It is possible to choose between 8 different patient chair colors and 7 cushion colors and fit the chair with standard or soft, hand-sewn cushions.

The cushions are fitted with XO Comfort fabric that allows the patient's skin to "breathe".

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ATTRACT PATIENTS AND EMPLOYEES

XO FLOW is the centerpiece of the clinic. The clean, timeless, functional design reflects the quality of the unit and stands as a testimony to professional judgment and good taste.

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When XO FLOW is shut down, the unit's water lines are emptied, and the suction lines are automatically disinfected. At power-up, the water lines are automatically disinfected whereafter the unit is ready for use.

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The Dashboard and Navigator are designed with reinforced glass fronts and thin protective steel frames. They can be used when wearing gloves and the Dashboard may be fitted with a glass screen protector.

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XO FLOW is available without cuspidor.

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The instrument holder can be removed, and the cover exchanged.

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PROTECTING THE PRACTITIONERS' HEALTH

SIT AND SEE WELL AT THE SAME TIME

The operators should sit in the upright balanced position to maintain the natural curvature of the spine in the lumbar region.

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There is lots of space under the chair's backrest for the operator's legs and feet. The chair has an automatic collision stop.

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The operator can position the light head so that the light beams are parallel to the direction of view.

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The neck rest can be placed in an upper jaw position and a lower jaw position.

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A unique feature of XO FLOW is that it is possible to sit well while also seeing well.

The right-handed dentist can work in positions from 9 through 12 o'clock.

The patient chair's neck rest makes it possible to place the patient's head in six different positions. This, combined with the flexibility of working in different positions, provides the operator with the best possible view of each tooth surface without bending or straining the neck, spine or upper body.

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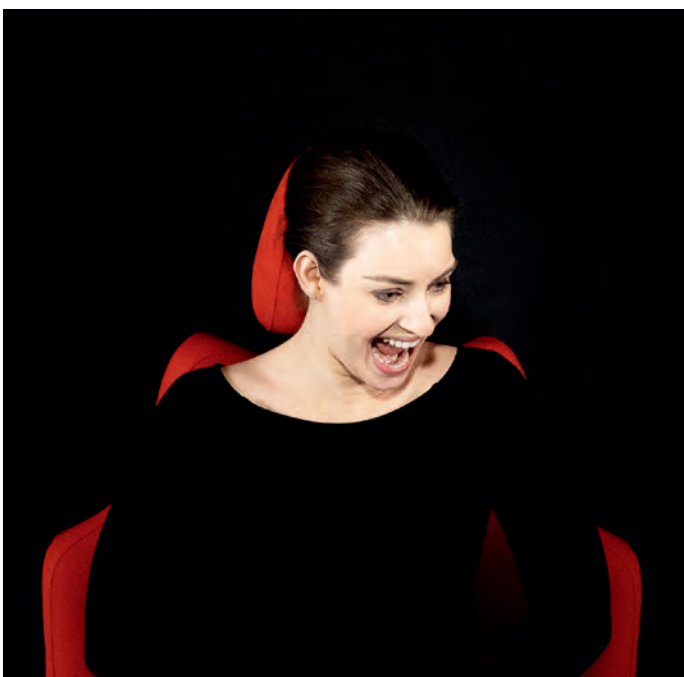
BALANCED INSTRUMENTS

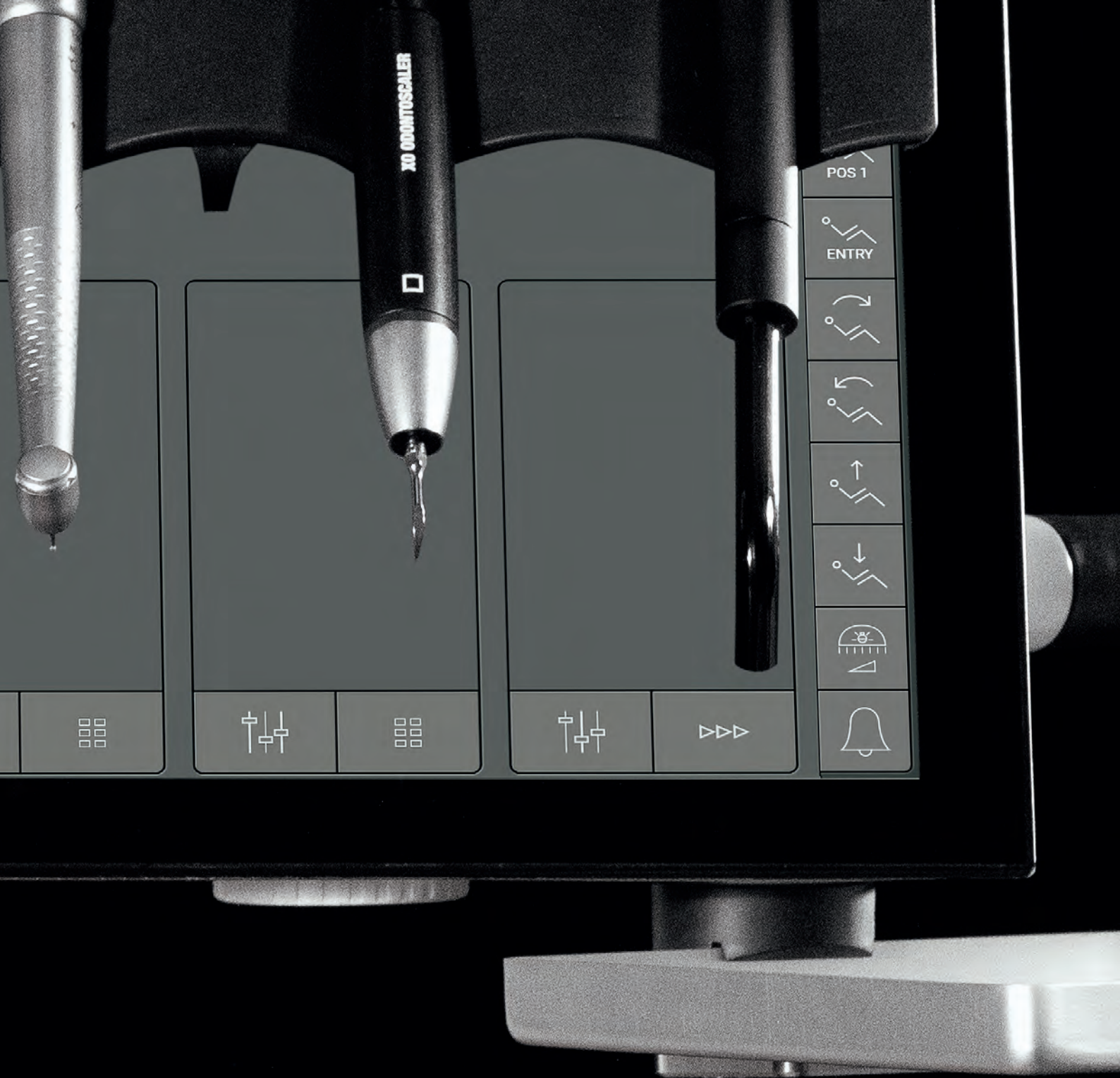
For perfect handling without strain on fingers, hands, arms and shoulders, the instruments are fitted with individually balanced suspensions.

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
One or two trays can be fitted in click-on holders under the instrument bridge.

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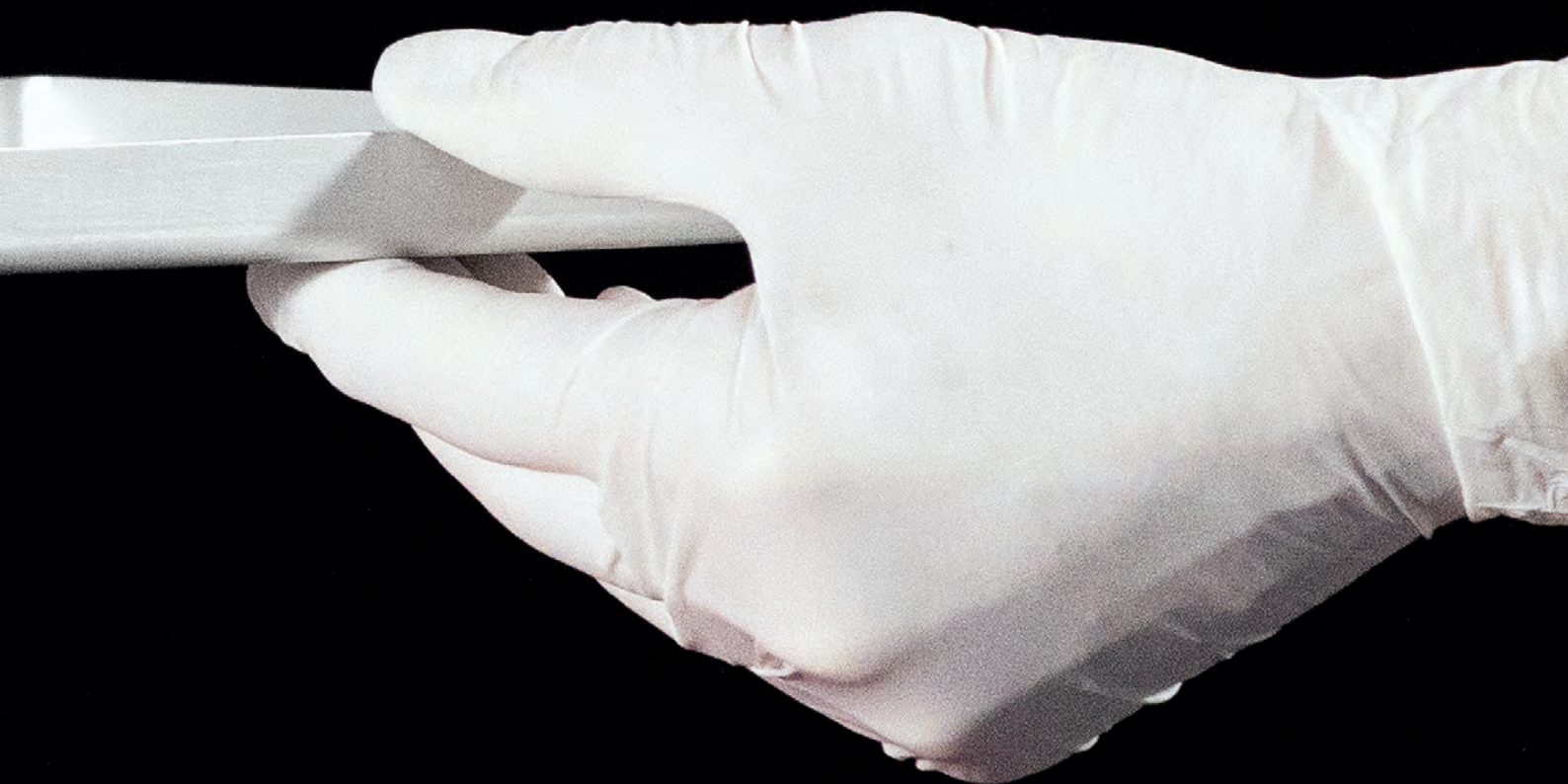The Ambidex suction hose holder may be placed in any position for working right-handed in duo, solo or left-handed.

After use, the suction hoses automatically reposition in the suction hose holder.

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WORK WELL

The dental team works well when both the dentist and the chairside assistant can operate the unit and reach the unit instruments without looking away from the oral cavity.

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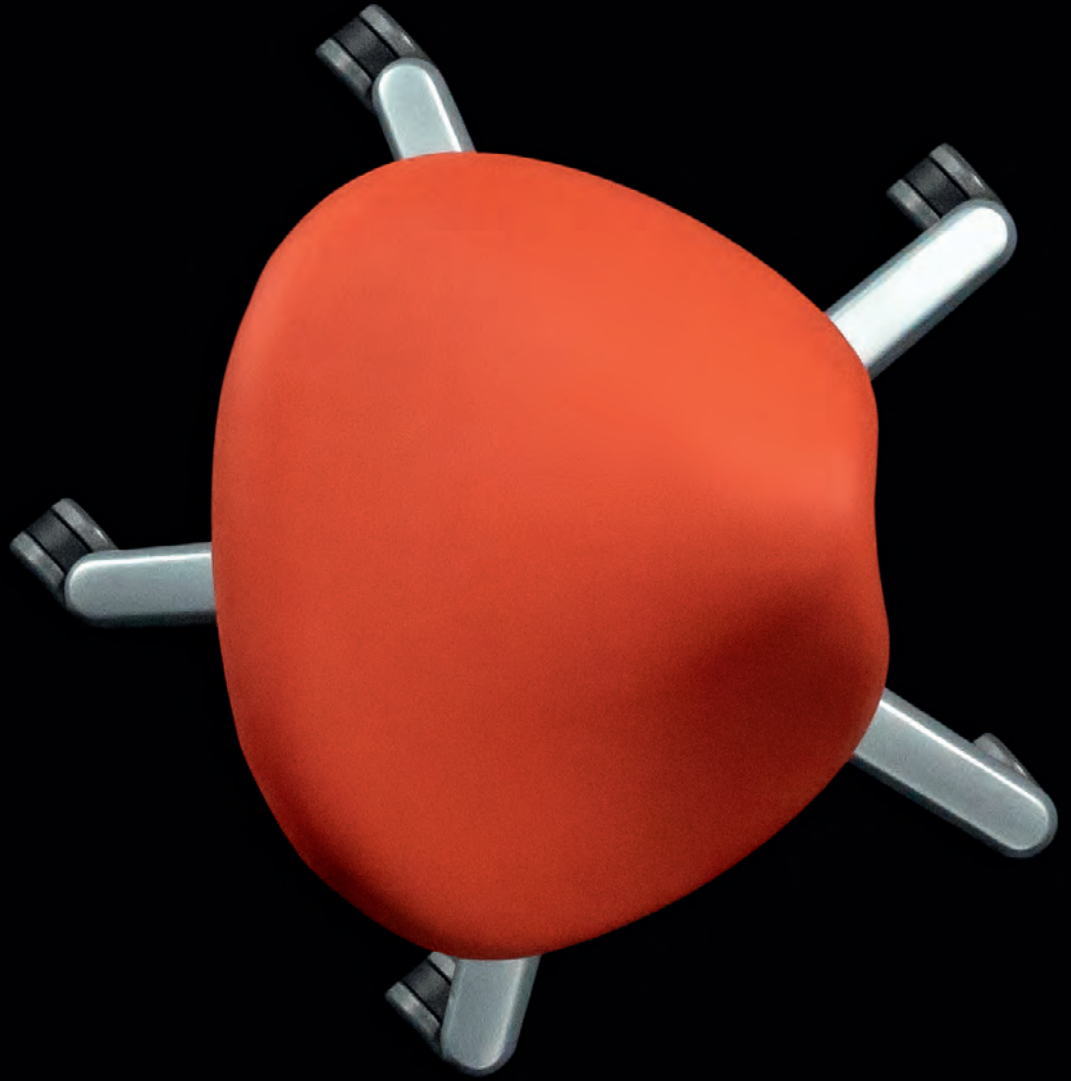
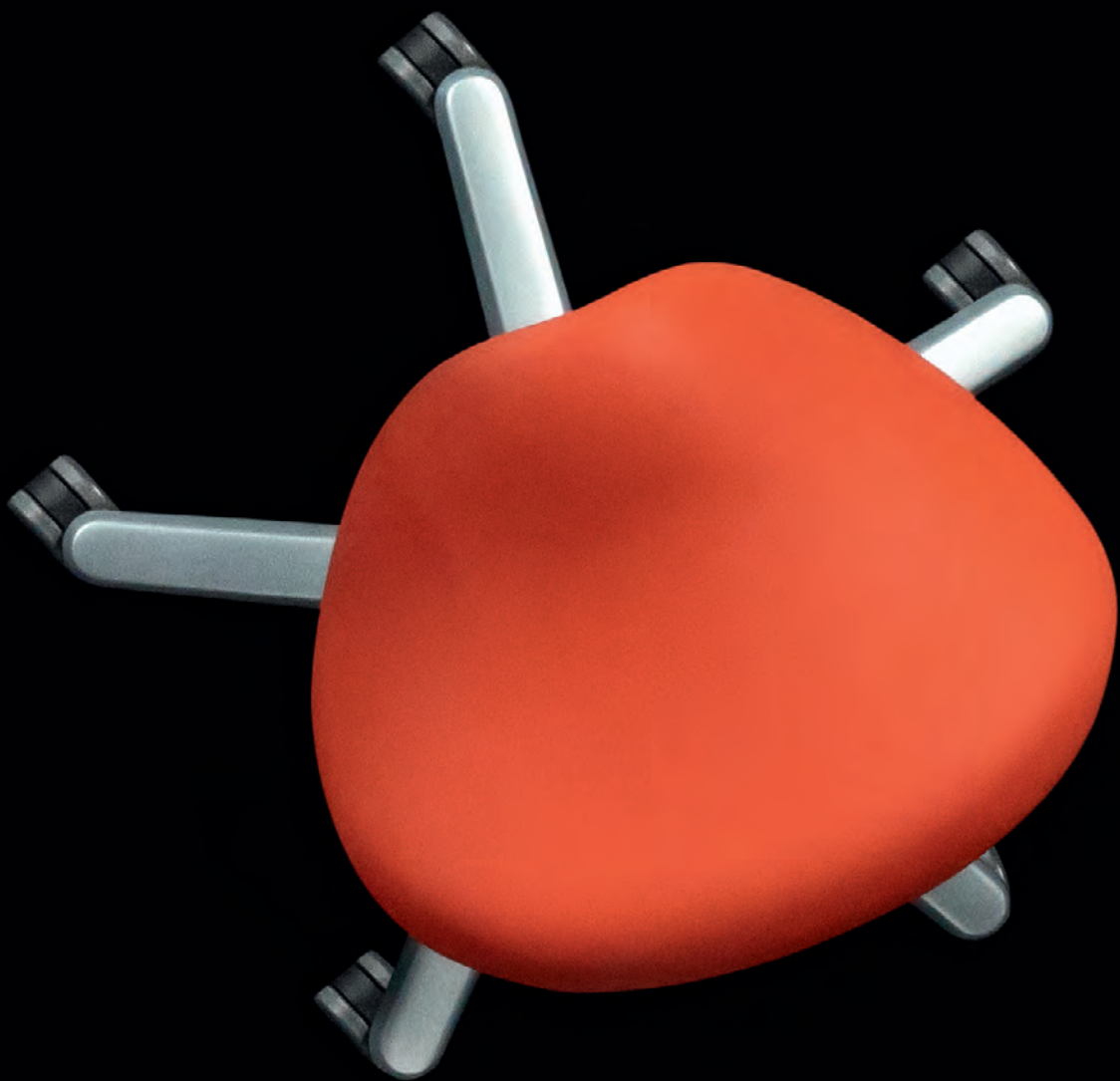
AMBIDEX SOLUTION

It takes less than 20 seconds to convert XO FLOW from right-handed to left-handed operation.

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ENSURING HIGH RETURN ON INVESTMENT

**WORK IN DUO
AND SAVE TIME**

XO FLOW improves productivity because the chairside assistant can easily prepare instruments and materials end exchange these with the dentist while handling the suction.

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WORK SOLO AND SAVE COSTS

With the suction hose holder in the solo position, the dentist can reach the suction hoses and be his or her own assistant.

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SAVE COSTS

Space-saving design

XO FLOW and XO WORKTOP only takes up 6 m² floor space.

This makes it possible to fit more treatment rooms into the premises and saves rental costs.

Low energy consumption

The unit automatically enters stand-by mode when it is not used.

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HIGH RETURN ON INVESTMENT

Low service costs

The service costs are minimized, as XO FLOW only needs preventive service every second year.

Low depreciation costs

The unit is built to last 20 years – therefore, also the depreciation costs are low.

A modular platform for the future

It is possible to place up to six instrument modules in any order and to exchange or add new modules later.

Software updates are offered free of charge.

XO FLOW is prepared for future needs and upgrades, such as new instrument modules with intraoral video camera or surgical motor, new features like reciprocal endo-function, new accessories like apex locator and peristaltic pump, new apps, extended workflow solutions, software and cloud computer integration, user statistics and remote technical diagnostics.

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HIGH UPTIME

XO FLOW is the culmination of 70 years of experience in developing, manufacturing and servicing high quality dental units. It is designed and manufactured in Denmark using materials and components of the highest quality standard.

In all valves controlling water, the valve piston is separated from the water to increase uptime and reduce service costs.

The unit's water system is protected by a water softener that effectively removes limescale.

Finally, XO FLOW's uptime is maximized due to the modular technique that confines possible faults to a single module which is easy to replace, if necessary.

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



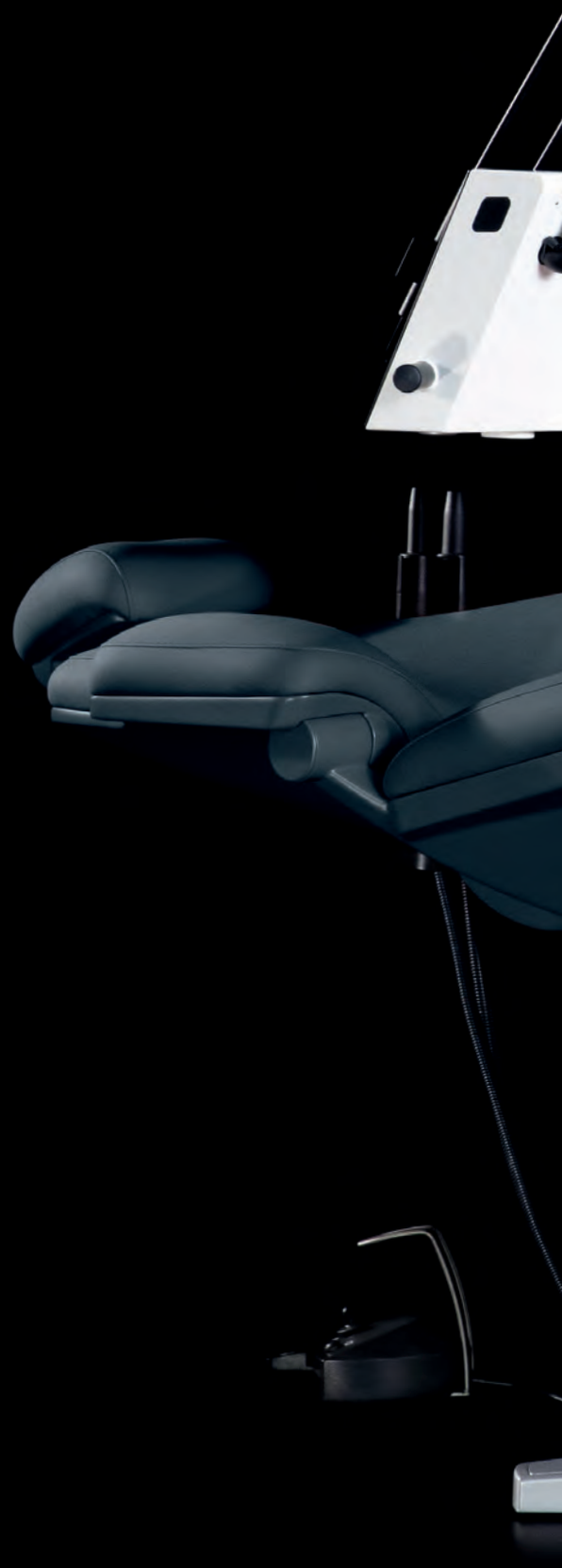
PERFORMING EXTRAORDINARY DENTISTRY

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SEE THE FILM

XO FLOW
Information film

English:

XO FLOW
Informationsfilm

Deutsch:

XO FLOW
Film d'information

Français :

C € 2460

All rights reserved.

XO® and XO FLOW® are registered
trademarks.

Patents: DK 1603478, FI 1603478,
FR 1603478, DE 60324334.7, IT
502009901695063, JP 4279260, US
7,739,125, CN ZL 201780009656.1, DK
3411001, FI 3411001, FR 3411001, DE 60
2017 010 673.9, IT 502020000030028, JP
6794459, UK 3411001, US 10,639,223.

Other patents pending.

© XO CARE A/S 2021.

Products shown in this catalogue are intended
for use by dental professionals for human
dentistry procedures.

See technical and legal product details at
xo-care.com.

XO FLOW, version 2022.

Some of the shown product features and
accessories will only be available during 2022.

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